

Troubleshooting

Pyon 2

错误代码类型：

1-1

描述:

No clear point of response found

措施/解决方案:

Test tips (breakage/wear/tightening torque)

1-1-1

描述:

Handpiece not recognised when starting Pyon 2

措施/解决方案:

Test handpiece connection to supply hose

1-1-1-1

描述:

Rinse supply hose thoroughly – handpiece recognised when starting Pyon 2

措施/解决方案:

Remove handpiece

1-1-1-1-1

描述:

> Foot control continuously active for longer than 15 mins > Transformer plugged-in during foot control operation (supply voltage restored) > Foot control operation before adjustment of power supply

措施/解决方案:

Deactivate foot control / restart

1-1-1-1-1-1

描述:

Electronics temperature error

措施/解决方案:

Allow to cool / restart

1-1-1-1-1-1-1

描述:

Overload at handpiece outlet

措施/解决方案:

Service technician

Intervall

描述:

Various system errors

措施/解决方案:

Service technician